

Instructions for use - Please read carefully!

Amocid®

Concentrate **Contact disinfection of medical devices (and surfaces)**

For professional use

COMPOSITION

100 g **solution** contains:
15.9 g Biphenyl-2-ol Sodium hydroxide

RANGE OF APPLICATION

- Disinfection and cleaning of medical devices (and surfaces) in **one working procedure only**
- Follow preparing instructions of the medical device manufacturer
- Disinfection of containers with tuberculous excretions (sputum, stool, urine)
- Laundry disinfection (incl. tb)

MODE OF APPLICATION / DOSAGE

Amocid® is a liquid concentrate with which any use dilution can be made by simply adding water (max: room temperature). Always fill up container with water first and then add **Amocid®**.

Disinfection of surfaces		
Prevention of nosocomial infections, routine disinfection (tb included)	2.50 %	- 1 h
Containers with tuberculous excretions	1.50 %	- 4 h
Cleaning by wiping accord. to VAH-procedure 2015 (incl. EN 16615), dirty conditions		
Bactericidal, levoricidal, tuberculicidal	2.50 %	- 1 h
	1.50 %	- 4 h
Control of epidemics according to § 18 of the law to protect against infections (IfSG)		
Surface disinfection	5.00 %	- 6 h
Laundry disinfection	1.00 %	- 12 h
Disinfection of sputum	5.00 %	- 4 h
Disinfection of stool	5.00 %	- 6 h
Disinfection of urin	5.00 %	- 2 h

Press 4 times on the pump (20 ml per pression) in a bucket containing 8 litres of water = 1 %

WARNINGS / STORAGE

- Keep disinfectants in a cool but frost free, well ventilated and dry place and out of reach of children!
- Wear adequate protective gloves, eye protection and suitable protective clothing when handling the concentrated product
- Do not use beyond expiry date.

PROPERTIES

- Control of bacteria (incl. tb) and fungus; also efficacious for disinfection of tuberculous excretions (albumin does not impair efficacy)
- Gentle disinfection of surfaces and laundry
- Very good wetting power, strong cleaning action without addition of detergents
- Makes stable solutions

SPECIAL PRECAUTIONS

a) Incompatibilities

Adding standard detergents to **Amocid**[®] can impair or neutralize its disinfecting properties.

b) Counter-measures in case of improper use

Should Amocid[®] come into contact with the mucous membranes (eyes, throat, stomach etc.) get medical attention immediately. Emergency measure: Dilute product without delay: if it has been taken orally, make patient drink plenty of water; if it has come in contact with the eyes, rinse thoroughly with plenty of water.

c) Miscellaneous

Amocid[®] is not appropriate for the disinfection of acryl surfaces.

d) Use **Amocid**[®] safely!

Always read the label and product information before use.

STANDARD PACKAGING / TRANSPORT UNITS

12 x 1 l bottle
2 x 5 l in cans

Article numbers:

1476

1505

Accessories:

Dosing pump for 5 l can
(1 stroke = 20 ml)

3135

2020/04/06_14

Lysoform Dr. Hans Rosemann GmbH

-Wilhelm-Str. 133, D-12247 Berlin
+49-30-77992-0 Telefax: +49-30-77992-219

www.lysoform.de, e-mail: kontakt@lysoform.de

Lysoform Schweizerische Gesellschaft für Antisepsie AG

Postfach 444, CH-5201 Brugg/Windisch
Telefon +41-56-441 69 81, Telefax +41-56-442 41 14

www.lysoform.ch