

Instructions for use - Please read carefully!

Fugaten®-Spray

(Lysoform®-Spray)

Ready for use

Disinfection-Spray based on alcohol for the rapid disinfection of surfaces, objects and medical devices

For professional use

COMPOSITION

100 g solution contains: 70 g Ethanol (denatured)

RANGE OF APPLICATION

For rapid disinfection of

- Small surfaces, objects and medical devices (e.g. surgical tables, examination chairs etc.)
- Surfaces difficult of access
- For all materials compatible with alcohol
- Product also exists without perfume, particularly adequate for use in the food industry

MODE OF APPLICATION / DOSAGE

Use undiluted only!

Spray the product from a distance of 30 cm on surfaces and objects until they are completely wet (max. 50 ml/m² of the room surface).

Allow product to dry before touching the disinfected surfaces/objects.

Disinfection by wiping according to the VAH procedure 2015 mechanical effect, clean conditions		
bactericidal, levurocidal	100%	60 s
Tests according to the directive of DVV/RKI		
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 2017	100%	30 s
rota virus	100%	30 s
adeno virus	100%	30 s
papova virus/polyoma virus/SV 40	100%	15 mn
EN-Norms		
EN 14348 (tuberculicidal, mycobactericidal), dirty conditions	100%	30 s
EN 14476 (polio virus), clean conditions	100%	5 mn
EN 14476 (MNV), clean conditions	100%	30 s
Virucidal	100%	15 mn

WARNINGS / STORAGE

- Fugaten®-Spray is easily inflammable!
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke. Max. quantity to be sprayed: 50 ml/ m² of the surface of the room
- Keep disinfectant in a dry, cool but frost-free and well-ventilated area, away from ignition sources and out of reach of children.
- Do not use beyond expiry date.

PROPERTIES

- Bactericidal (incl. tb), levurocidal, virucidal
- Rapid action and drying after spraying on surfaces and objects
- Fresh odour; deodorizes the surrounding air
- Good wetting power
- No adverse effect on most materials
- Ready to use product in refillable, polyethylene bottles with spray-pump

SPECIAL PRECAUTIONS

a) Counter-measures in case of improper use

Fugaten® Spray is irritating by contact with the mucous membranes (eyes, throat, stomach, bowels etc.). In case of such irritations dilute product immediately. If it has been taken orally, make patient drink plenty of water; if it has been in contact with other mucous membranes, rinse thoroughly with plenty of water. In each case, get medical attention.

b) Miscellaneous

Do not apply on materials which are not compatible with alcohol (coated, pickled surfaces, acrylic glass etc.).

c) Use Fugaten® Spray safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

STANDARD PACKAGING / TRANSPORT UNITS

	Article No:
10 x 500 ml bottle <u>with</u> spray pump	2058
12 x 1000 ml bottle <u>with</u> spray pump	2292
12 x 1000 ml bottle <u>with</u> spray pump (without perfume)	2272
2 x 5 litre canister	9825
2 x 5 litre canister (unscented)	9823

Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D-12247 Berlin

Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

www.lysoform.de, e-mail: kontakt@lysoform.de

Lysoform Schweizerische Gesellschaft für Antisepsie AG

Postfach 444, CH-5201 Brugg/Windisch

Telefon +41-56-441 69 81, Telefax +41-56-442 41 14

www.lysoform.ch

2020/04/06-18