

Instructions for use - Please read carefully

## Fugaten®

### For professional use

**Ready-for-use disinfectant based on alcohol for a rapid disinfection of surfaces, objects and medical devices**

### COMPOSITION

100 g solution contains: 70 g Ethanol (denatured)

### RANGE OF APPLICATION

For a rapid disinfection of

- Small surfaces and objects (e.g. operating tables, examination chairs etc.)
- other alcohol-resistant materials

### MODE OF APPLICATION / DOSAGE

Use undiluted only!

Apply **Fugaten®** on objects and surfaces and rub with a dry cloth. Surfaces and objects have to be fully wetted with the product.

<b>Disinfection by wiping according to the VAH procedure 2015, mechanical effect, clean conditions</b>		
bactericidal, levurocidal	100%	60 s
<b>Tests according to the directive of DVV/RKI</b>		
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 2017	100%	30 s
rota virus	100%	30 s
adeno virus	100%	30 s
papova virus/polyoma virus/SV 40	100%	15 mn
<b>EN-Norms</b>		
EN 13624 (fungicidal), clean and dirty conditions	100%	1 mn
EN 14348 (tuberculicidal, mycobactericidal), dirty conditions	100%	30 s
EN 14476 (polio virus), clean conditions	100%	5 mn
EN 14476 (MNV), clean conditions	100%	30 s
Virucidal	100%	15 mn

### WARNINGS / STORAGE

- **Fugaten®** is easily inflammable!
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke.
- Max. quantity to be released: 50 ml/ m<sup>2</sup>.
- Keep disinfectant under cool, dry and frostfree conditions, away from ignition sources and out of reach of children.
- Do not use beyond expiry date.

### PROPERTIES

- Bactericidal (incl. tb) – levuricidal – virucidal
- Rapid action and drying after disinfection of surfaces and objects
- Fresh odour
- Good wetting power
- Good compatibility with most materials

### SPECIAL PRECAUTIONS

a) Counter-measures in case of improper use

**Fugaten®** is irritating by contact with the mucous membranes (eyes, throat, stomach, bowels etc.). In case of such irritations dilute product immediately. If it has been taken orally, make patient drink plenty of water; if it has been in contact with other mucous membranes, rinse thoroughly with plenty of water. In each case, get medical attention.

b) Miscellaneous

Do not apply on materials which are not compatible with alcohol (coated, pickled surfaces, acrylic glass etc.).

c) Use **Fugaten®** safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

Translated by Lysoform Dr. Hans Rosemann GmbH, Berlin - Germany

**STANDARD PACKAGING / TRANSPORT UNITS**

12 x 1 l bottle  
12 x 1 l bottle (unscented)

**Article No:**

9685  
2299

**Lysoform Dr. Hans Rosemann GmbH**

Kaiser-Wilhelm-Str. 133, D-12247 Berlin  
Tel.: +49-30-77992-0   Telefax: +49-30-77992-219  
[www.lysoform.de](http://www.lysoform.de), e-mail: [kontakt@lysoform.de](mailto:kontakt@lysoform.de)

**Lysoform Schweizerische Gesellschaft für Antisepsie AG**

Postfach 444, CH-5201 Brugg/Windisch  
Telefon +41-56-441 69 81, Telefax +41-56-442 41 14  
[www.lysoform.ch](http://www.lysoform.ch)

2020/04/06-11