

# Hospisept® Tuch (wipes)

Disinfecting and cleaning wipes for  
surfaces, medical equipment and medical devices

## Product information

Please read carefully!

### Composition

1 moist **wipe** contains on average:  
1.1 g Propan-1-ol, 0.32 g Ethanol 96 % (active ingredients)  
Butan-2-on, Dibutyl Adipate, Purified Water

### Range of application

- Disinfection and cleaning of small surfaces, disinfection of medical devices, for example of contact glasses for diagnostic purposes, of dental equipment etc., and antimicrobial pre-cleaning of endoscopes
- To be used where it is not possible to spray the product or when rapid drying is important.
- Hygienic hand disinfection

### Mode of application / dosage

- Wipe the surface (always make sure that it is completely wetted with the disinfectant!) and allow product to work until dry (about 5 minutes).
- Hygienic hand disinfection: 30 seconds. Wet hands completely with product. (use 2 wipes if necessary).
- Virucidal against enveloped viruses, incl. HBV/HCV/HIV and  
rota virus and noro virus (MNV): 30 sec  
adenovirus: 1 mn  
polyomavirus (SV 40): 5 mn

\* according to recommendation of RKI 2017

### Warnings / storage

- Inflammable (flash point acc. to DIN 51 755: 26 °C).
- Keep this medical device in a cool – but frost free – and dry place, away from ignition sources and out of reach of children.
- Avoid contact of wipe with the eyes.
- Do not use on alcohol-sensitive plastics, e.g. acrylic glass.
- Shelf-life after opening bag: 6 months.
- Do not use beyond expiry date.

### Properties

- Comprehensive range of action: against bacteria (incl. tb), fungi as well as enveloped viruses and rota viruses
- Pre-cut, to tear off individually
- Ready for use
- Well tolerated by the skin, contains no surfactants, no perfumes
- Whereas other disinfecting and cleaning wipes for surfaces are to be used wearing gloves according to BGR 250/TRBA 250, Hospisept® wipes can be used by unprotected hands
- Suitable for emergency kits (in ambulances, rescue helicopters)
- No hazardous waste left after use

Certified and mentioned in the relevant lists.

### Standard packaging / transport units:

6 x container of 100 wipes  
6 x refill bags of 100 wipes  
15 x Hospisept® wipes, packed individually, in small cardboard boxes

### Article No:

3585  
3575  
3595

## Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D – 12247 Berlin  
Telefax 030 - 77 99 2-269, Info.-Telefon 0700LYSOFORM  
[www.lysoform.de](http://www.lysoform.de) , e-mail: [kontakt@lysoform.de](mailto:kontakt@lysoform.de)

2019/10/23-17

