

Instructions for use - Please read carefully!

# Lysoformin® Plus

**Concentrate**                      **Disinfecting and cleaning concentrate for surfaces, equipment and medical devices**

for professional use

## COMPOSITION

100 g **solution** contains:  
 8.9 g Polyhexamethylene biguanide . HCl, 23.2 g Bis-(3-aminopropyl)dodecylamine,  
 5.0 g Didecyl Dimethyl Ammonium Chloride  
 Contains: (R)-p-Mentha-1,8-diene

## RANGE OF APPLICATION

- One-step application for the disinfection and cleaning of surfaces, and equipment (incl. medical devices) in case of bacterial infections.
- Use according to the preparation instructions of the manufacturer of the medical devices.
- Control of nosocomial infections
- Disinfection and hygienic cleaning of all sanitary equipment.

## MODE OF APPLICATION / DOSAGE

**Lysoformin® Plus** is a liquid concentrate with which any use-dilution required can be prepared by simply adding water (at max. room temperature). Always fill container with tap water first, then add disinfectant.

<b>Disinfection by wiping according to the VAH procedure 2015, dirty conditions</b>		
bactericidal, levurocidal	2.00%	5 mn
	1.00%	15 mn
	0.75%	30 mn
	0.50%	1 h
	0.25%	4 h
<b>Tests according to the directive of the DVV/RKI</b>		
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 2017*	0.50%	15 mn
Papova virus / Polyomavirus / SV40*	0.50%	30 mn
<b>EN-Norms</b>		
virucidal against enveloped viruses according to EN 14476 (clean and dirty conditions)	0.50%	5 mn
EN 14476 rota virus (clean conditions)	0.50%	5 mn
EN 14476 MNV (clean conditions)	2.00 %	2 h
EN 14476 SV40 (clean conditions)	2.00 %	5 mn
EN 14476 SV40 (dirty conditions)	2.00 %	15 mn
EN 16615 (dirty conditions)	2.00 %	5 mn

\* DVV 2015 (suspension test)

## Lysoform – Hygiene with System

Concentration	0.25 %	0.5 %	1.0%
<b>Dosing system</b> (for 8 l of water)			
Dosing pump (1 stroke ≙ 20 ml)	1 stroke	2 strokes	4 strokes

**Important:** Always fill up container with water (at room temperature) and then add the disinfectant. The amount of water to be used is calculated by subtracting the amount of disinfectant needed to obtain the concentration wished for from the total amount of solution needed.

Translated by Lysoform Dr. Hans Rosemann GmbH, Berlin - Germany

## WARNINGS / STORAGE

- Keep disinfectants in a dry, cool but frost free and well ventilated place and out of reach of children.
- Wear suitable protective clothing, adequate protective gloves and eye protection when handling the concentrated product.
- Do not use beyond expiry date.

## PROPERTIES

- Contains no aldehydes, no phenols, no phosphates; free of oxidizing compounds
- All-round disinfecting efficacy, good compatibility with surfaces and materials
- Good cleaning power without addition of detergents even in case of heavy soil by organic substances like fat, albumin, serum
- Fresh scent
- Resists hard water
- Biodegradable

## SPECIAL PRECAUTIONS

a) Incompatibilities:

Adding detergents to **Lysoformin® Plus** can impair or neutralize its disinfecting properties.

b) Counter-measures in case of improper use:

Should **Lysoformin® Plus** come into contact with the mucous membranes (eyes, throat, stomach etc.) due to improper use get medical attention immediately.

As emergency measure, dilute product without delay: If it has been swallowed, drink **plenty** of water; after contact with the eyes, rinse thoroughly with **plenty** of water.

c) Miscellaneous

Objects/surfaces which come into contact with foodstuff must be rinsed with drinking water at the end of the contact/exposure time.

d) Use **Lysoformin® Plus** safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

## STANDARD PACKAGING / TRANSPORT UNITS

6 x 1000 ml bottles

2 x 5 litre canisters

## Article numbers:

1865

1869

## Accessories

Dosing pump for 5 l canister (1 stroke  $\cong$  20 ml)

3135

## Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D-12247 Berlin

Tel.: +49-30-77992-0    Telefax: +49-30-77992-219

[www.lysoform.de](http://www.lysoform.de), e-mail: [kontakt@lysoform.de](mailto:kontakt@lysoform.de)

## Lysoform Schweizerische Gesellschaft für Antisepsie AG

Postfach 444, CH-5201 Brugg/Windisch

Telefon +41-56-441 69 81, Telefax +41-56-442 41 14

[www.lysoform.ch](http://www.lysoform.ch)

2020/08/31\_09