

Instructions for use - Please read carefully!

Lysoformin® Plus-Schaum

Ready for use

fast-acting disinfecting and cleaning foam for medical inventory

For professional use

Composition

100 g Lysoformin® Plus-Schaum contains:
0.18 g Polyhexamethylene biguanide · HCl,
0.46 g Bis-(3-aminopropyl) dodecylamine,
0.10 g Didecyl dimethyl ammonium chloride, fragrances

Range of application

Fast-acting disinfectant for medical inventory within the meaning of the Medical Devices Act (MPG). Also suitable for other water-resistant surfaces.

- Disinfection and cleaning of surfaces (including medical devices) in a single operation
- Application according to reprocessing instructions of the medical device manufacturer

Mode of application / dosage

Lysoformin® Plus-Schaum is ready for use. Spray foam on the surfaces and objects to be disinfected and allow to act. Then eliminate with a clean, damp cloth. Product can also be applied undiluted on a one-way cloth which will be used to wipe the surfaces to be disinfected and cleaned.

Contact time: **5 minutes**

Virucidal against enveloped viruses*

Incl. HBV/HCV/HIV **5 min**
Polyomavirus (SV 40) **15 min**

*according to recommendation of Robert Koch Institute Berlin/ RKI 2017

Warnings / storage

- Do not inhale spray or foam; If necessary wear respiratory protection with a suitable particle filter.
- Wear adequate protective gloves and protective clothing when handling the product.
- Do not use beyond expiry date.
- Keep disinfectants in a dry, cool but frost free and well ventilated area and out of reach of children.

Properties

- control of bacteria, fungus, enveloped viruses
- optimal wetting action of the surfaces and thus gentle and reliable disinfection
- easy handling, ready to use
- fresh odour

Special precautions

a) Counter-measures in case of improper use:

Should **Lysoformin® Plus-Schaum** come into contact with the mucous membranes (eyes, throat, stomach, etc.) through improper use get medical attention immediately. As an emergency measure: Dilute product immediately: if it has been swallowed, drink **plenty** of water; after contact with the eyes, rinse thoroughly with **plenty** of water.

c) Miscellaneous:

Rinse objects/surfaces that come in contact with food with drinking water after the recommended exposure time e.g. after completed disinfection.

d) Use disinfectants safely. Always read the label and product information before use.

Standard packaging / transport units

12 x 1 litre (bottle) with foam sprayer
2 x 5 litres (canister)

article numbers

2505
2509

Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Telefax 030 - 77 99 2-269, Info.-Telefon 0700LYSOFORM
www.lysoform.de , e-mail: kontakt@lysoform.de

2019/11/14_6

Translated by Lysoform Dr. Hans Rosemann GmbH, Berlin - Germany