

Instructions for use - Please read carefully!

Lysoformin® spezial

Concentrate

Wipe Disinfection and cleaning of surfaces and medical devices

For professional use only!

COMPOSITION

100 g solution contains:

2.9 g Polyhexamethylene biguanide · HCl, 9.8 g Didecyl dimethyl ammonium chloride

RANGE OF APPLICATION

Disinfection and cleaning

- of surfaces and equipment, of textiles like mops, floor cloths etc. in a single working step
- for the control of nosocomial infections
- in kitchen areas
- in bathing facilities, saunas, laundries etc.
- of solarium benches.

MODE OF APPLICATION / DOSAGE

Lysoformin® spezial is a liquid concentrate with which any use-dilution required can be made by simply adding water (at max. room temperature). Always fill container with tap water first and then add Lysoformin® spezial.

Any objects coming into contact with food has to be rinsed with drinking water after disinfection/cleaning.

Surface disinfection and cleaning		
Prevention of nosocomial infections and routine disinfection	0.25 %	4 h
	0.50 %	1 h
	0.75 %	30 mn
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 2017	0.50 %	30 mn
Rota virus	0.50 %	1 mn
bactericidal – Legionella pneumophila	0.25 %	5 mn
Textilien (mops, floor cloths etc.)		
	0.75 %	4 h

Lysoform – Hygiene with System

Concentration	0,25 %	0,5 %	0,75%
Dosing-system (for 8 l of water)			
Dosing-bag (1 bag ≙ 40 ml)		1 bag	
Dosing pump (1 stroke ≙ 20/30 ml)	1 stroke à 20 ml	2 strokes à 20 ml	2 strokes à 30 ml or 3 strokes à 20 ml
Lysoform dosing device DH 91 – no electricity needed.	Adjust once to obtain the desired concentration		

Important: Always pour water (at room temperature) into the container first and then add the disinfectant.

The necessary amount of water is calculated by subtracting the amount of disinfectant needed to obtain the concentration wished for from the total amount of solution needed.

WARNINGS AND STORAGE

- Keep disinfectants in a dry, cool but frost free and well ventilated place and out of reach of children
- Wear suitable protective clothing, adequate protective gloves and eye protection when handling the concentrated product.
- Do not use beyond expiry date.
- Shelf-life of the ready-for-use solutions without residue of organic material * and when stored properly (in a cool and dry place) in clean and tightly closed containers: 4 weeks.

PROPERTIES

- control of bacteria (incl. salmonella), fungi, enveloped viruses and rota virus
- good compatibility with surfaces and equipment
- contains no aldehydes, no phenols, no phosphates, no oxidising compounds
- good cleaning power without addition of detergents
- fresh, non clinging scent
- biodegradable

SPECIAL PRECAUTIONS

a) Incompatibilities:

Adding the common detergents to Lysoformin[®] spezial can impair or neutralize its disinfecting properties.

b) Counter-measures in case of improper use:

Should Lysoformin[®] spezial come into contact with the mucous membranes (eyes, throat, stomach etc.) seek medical attention immediately. In all cases, dilute the product without delay: If swallowed, drink plenty of water and ring up emergency call; after contact with the eyes, rinse thoroughly with plenty of water.

c) Use disinfectants safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

STANDARD PACKAGING / TRANSPORT UNITS

	Article numbers
200 x 40 ml dosing bags = 8 litres in the cardboard box	9287
12 x 1 litre bottles	9291
2 x 5 litres canisters	9325
1 x 20 litres canisters	9320

Accessories:

Dosing pump for 5 l canister (1 stroke \cong 20 ml) 3135

Dosing pump for 5 l canister (1 stroke \cong 30 ml) 3136

Hygiene Technique:

Lysoform Dosing Device 9174

“ period of time during which the proven efficacy of the ready-for-use disinfectant solution is not affected, after preparation at the concentration according to the recommendations of the manufacturer and if it has been stored during the period of time mentioned according to the recommendations of the manufacturer and left to stand unused.

Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D-12247 Berlin

Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

www.lysoform.de, e-mail: kontakt@lysoform.de

Lysoform Schweizerische Gesellschaft für Antiseptie AG

Postfach 444, CH-5201 Brugg/Windisch

Telefon +41-56-441 69 81, Telefax +41-56-442 41 14

www.lysoform.ch

2020/09/01_18