

Instructions for use - Please read carefully!

## Virucidal and skin compatible!

# NEOSEPTIN®

alcoholic hand disinfectant  
disinfection of small surfaces and medical devices.

Ready for use!

For professional use!

### COMPOSITION

100 g Neoseptin® contains: 79,9 g Ethanol 96%

### RANGE OF APPLICATIONS

- Control of bacteria (tb included), fungus and viruses
- Disinfection of hands as well as of small surfaces and medical devices

### MODE OF APPLICATION / DOSAGE

Use externally and **undiluted** only! Disinfection of small surfaces and medical products: Apply **Neoseptin®** and rub with a dry cloth. Surfaces and objects have to be completely wet with the product (max. 50 ml/m<sup>2</sup>)

<b>Hand disinfection according to the standard procedures of the VAH of 2015 under clean and dirty conditions</b>		
bactericidal, levurocidal	30 s	
<b>Tests according to the DVV/RKI directive</b>		
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 3/2017	30 s	
<b>EN-norms (clean conditions)</b>		
Virucidal against enveloped viruses PLUS according to EN 14476 (adeno virus and MNV)	30 s	
Virucidal according to EN 14476	30 s	
<b>EN norms hygienic and surgical hand disinfection</b>		
EN 1500	30 s	
EN 12791	90 s	
<b>Disinfection by wiping according to the VAH standard procedures of 2001/2002, with mechanical effect, clean and dirty conditions</b>		
Bactericidal, levuricidal	1 mn	

### WARNINGS / STORAGE

- Do not apply on open wounds and on mucous membranes!
- **Neoseptin®** is easily inflammable.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke.
- Store this medical device in a dry, cool but frost-free and well ventilated area and out of reach of children.
- Do not use beyond expiry date.

### PROPERTIES

- Contains no perfume
- Comprehensive spectrum of efficacy
- Reliable action in the presence of protein, serum and blood
- Well tolerated by the skin even with frequent use (moisturising and re-fatting properties)
- leaves the skin with a pleasant smooth feeling.

**SPECIAL PRECAUTIONS**

**Neoseptin**<sup>®</sup> is a medical device of class II A: CE 0482.

Use disinfectants safely. Always read the label and product information before use.

Certified and listed in the relevant reference documents

**STANDARD PACKAGINGS / TRANSPORT UNITS**

	<b>Article No.</b>
12 x 500 ml bottle	2700
6 x 1 l litre bottle	2705
15 ml SaniStick with spraying nozzle	2708

**Lysoform Dr. Hans Rosemann GmbH**

Kaiser-Wilhelm-Str. 133, D-12247 Berlin

Tel.: +49-30-77992-0    Telefax: +49-30-77992-219

[www.lysoform.de](http://www.lysoform.de), e-mail: [kontakt@lysoform.de](mailto:kontakt@lysoform.de)

**Lysoform Schweizerische Gesellschaft für Antisepsie AG**

Postfach 444, CH-5201 Brugg/Windisch

Telefon +41-56-441 69 81, Telefax +41-56-442 41 14

[www.lysoform.ch](http://www.lysoform.ch)

2020/01/23\_6